whiteClass

DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Authorization of Functioning MS P5X44XY0XX28 - CNPJ/Tax ID: 05.106.945/0001-06 - BRAZILIAN INDUSTRY - Registration at ANVISA Nº 80172310039 Technical Authority: Friedrich Georg Mittelstädt - CRQ: 13100147-SC - Brand: FGM®

Rev: 01

Irritante

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At-Home Bleaching System

Instructions for the Patient

Read carefully all the information in this product information sheet before using the product. Keep it for reference until total consumption of the product.

Instructions for Use

Fig 1. Press the syringe plunger and place a small drop of the gel in into every compartment of the bleaching tray from teeth undergoing treatment. Usually one small drop for each tooth is enough to cover the whole buccal surface.


Fig 2. Seat the tray, with the gel, around the teeth with a slight pressure with the aim to distribute gel around all dental surfaces.

Fig 3. Wipe away excess bleaching gel in mouth with a dental brush or with the fingers.

Fig 4. Employ the bleaching gel according to your dentist's instructions. In case of any discomfort or any other important finding inform your dentist immediately.

Fig 5. Rinse well the bleaching tray before storage and before use. Store the bleaching gel in a cold place. Avoid prolonged exposure to light. Discard the empty syringes adequately.

Step - by - Step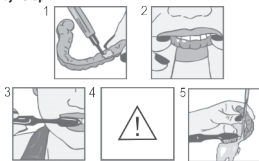

Important: Follow strictly your dentist's instructions. The treatment success depends on the adequate compliance with the treatment guidelines. In case the bleaching tray is removed it should be reloaded before seating it again in the mouth.

Important Information

During the bleaching treatment there might be tooth sensitivity due to temperature variations, mainly to cold. This tooth sensitivity can be treated and minimized with some treatments that your dentist can prescribe. Contact your dentist if the sensitivity becomes uncomfortable. Some patients might also have some slight irritation of the gingival tissue, throat, tongue and lips which is usually due to excess of bleaching gel in the bleaching tray. Contact your dentist whenever you notice any of the above reactions or problems. DO NOT SELF PRESCRIBE.

It is recommended to avoid the ingestion of acid drinks or food during bleaching as they might increase the tooth sensitivity triggered by the

bleaching procedure. Chromatic drinks and foods should be avoided to not jeopardize the bleaching process. Follow strictly your dentist's instructions.

Precautions and Contra-indications

The product should be indicated and used only under dentist's supervision and control.

The use during pregnancy or lactation is not recommended as well as in patients under 15 years old.

The product is not recommended for patients who have recently undergone gingival surgery, who has gingival tissue inflammation or some allergic reaction to any of the components of the product.

Conservation and Storage

Store the product at temperature between 5°C and 25°C / 41°F and 77°F. Do not freeze the product. Keep out of direct sunlight.

Warnings:

Do not use the product after the expiration date. Discard the product following the legislation of your country. Do not reuse the empty packing. Keep the product out of children's reach.

This product was manufactured for dental use only and it should be manipulated according to the instructions. The manufacturer is not responsible for damage caused by other types of use or by incorrect manipulation. Furthermore, the user is obligated to verify, before the utilization and under his/her responsibility, if this product is compatible with the desired utilization, especially when such use is not mentioned in the instructions. Data descriptions do not constitute any type of warranty and, therefore, are not legally binding.