whiteness simple

DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Authorization of Functioning MS P5X44XY0XX28 - CNPJ/Tax ID: 05.106.945/0001-06 - BRAZILIAN INDUSTRY - Registration at ANVISA Nº 80172310046 Technical Authority: Friedrich Georg Mittelstädt - CRQ: 13100147-SC - Brand: FGM®

Rev: 01

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## EN Instructions Manual

### Dental Bleaching for supervised home use

#### Instructions for the Patient

*Read carefully all the information in this product information sheet before using the product. Keep it for reference until total consumption of the product.*

This Instruction Manual has only orientable purpose about the basic procedures of home bleaching, and does not replace the necessity of instructions from the professional responsible with the treatment. The DDS has an instruction manual with detailed information about the product and the procedure, and it should be consulted whenever necessary.

#### Instructions for Use


Pic1. Adjust the applicator tip in the whitening gel syringe. Press the plunger of the syringe and apply the gel in the recesses tray corresponding to each dental space to be cleared. A small drop is enough for a tooth.

Pic2. Fit the tray on teeth and slightly press to engage them with gel.

Pic3. With your finger or toothbrush remove the excess gel that overflows to the gums.

Pic4. Use the gel as recommended by your dentist. Communicate to the dentist any discomfort or important observation whenever it is needed.

Pic5. Wash the tray well before storing it and before use. Store the gel in a cool place. Avoid prolonged exposure of the syringe to light. Dispose appropriately the empty syringes.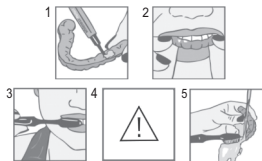

Important: Important: Follow the instructions. The success of treatment depends on the smooth conduct of the procedure.

#### Important Information

Tooth Sensitivity: the occurrence of sensitivity during bleaching is variable and their frequency and intensity differ from patient to another. To minimize this undesirable effect, see the responsible dentist for treatment; he can make use of desensitizing agents that bring comfort to the process. Report to the dentist immediately if there is too much sensitivity.

Some patients may experience irritation in the gums, throat, tongue or lips, usually as a result of excessive use of gel in the tray. Communicate your dentist if you notice any problem or different reactions. **DO NOT SELF-MEDICATION.**

It is recommended to avoid intake of acidic foods or drinks during bleaching because these can cause increased sensitivity during bleaching. Beverages or strongly colored foods should also be avoided. Follow carefully the instructions of your dentist.

#### Precautions and Contra-indications

The product should be indicated and used only under supervision and control of a dentist.

The use during pregnancy or lactation is not recommended as well as in patients under 15 years old.

The product is not recommended for patients who have recently undergone gingival surgery, patients with gingival tissue inflammation or some allergic reaction to any of the components of the product.

#### Conservation and Storage

Store the product at temperatures between 5 and 27 ° C. Do not freeze the product. Protect from direct sunlight.

#### Warnings

Do not use the product if it is out of shelf life. Follow the laws of your country for the product disposal. Maintain outside the reach of the children.

This material was developed only for dental use and must be handled according to the instructions. The manufacturer is not liable for damages caused by other uses or incorrect handling. In addition, the user must prove, before the use and under his or her responsibility if this product is compatible with the desired utilization, especially when this use is not indicated in these operating instructions. Data descriptions do