Rev: 03

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#### Professional use only

Read all the information in this manual carefully before using the product. Keep the manual for later reference at least until the product is finished or until there is no more interaction between the product and its last patient.

#### Product description

Opalis is a nanohybrid, radiopaque, light-curing resin with fluorescent and opalescent properties, high polishing power and several options of shades and translucence. It is an aesthetic restorative material used for restorations of anterior and posterior teeth.

Opalis was carefully developed to reproduce the polychromatism of teeth in great detail, allowing a longer work period (reduced sensitivity to light) with great mechanical properties.

Opalis Odontopediatria has the same composition, use and indications as Opalis. However, the shades and opacity of Opalis Odontopediatria allow for better aesthetics of deciduous teeth.

#### Presentation

Refill: 1 syringe with 1 resin shade.

#### Basic Kit: 6 syringes

Shades: EA2, EA3, EA3.5, DA2, DA3 and T-Neutral

Kit Intro : 4 syringes, 1 Condac 37% and 1 Ambur

Shades: EA2, EA3 and DA2.

#### Clinical Kit: 15 syringes

Shades: EA1, EA2, EA3, EB2, DA1, DA2, DA3, DB2, D-Bleach,

T-Blue, T-Neutral, T-Yellow, E-Bleach H, Opaque Pearl and VL.

#### Basic composition

Opalis is a nanohybrid resin composed by a monomeric matrix containing Bis (GMA), Bis (EMA), UDMA and TEGDMA. The fillers are a careful combination of Barium-Aluminum, silanized silicate and nanoparticles of silicon dioxide, carboronoroquione as photoinitiator, accelerators, stabilizers and pigments.

The composite has particles in the range 40nm to 3.0 microns with an average particle size of 0.5 microns, the total filler content of 78.5 wt% to 79.8 volume% and 57% to 58% inorganic fillers.

The distribution of the sizes of the particles allows an adequate filling of the resin which contributes for its high mechanical resistance as well as resistance to wear. Such characteristics are necessary for restorations of posterior teeth and for easy polishing, generating a restoration with flat surface and high brightness for anterior teeth.

#### Product indication

Opalis is indicated for restorations of anterior and posterior teeth allowing the reproduction of the optical characteristics of the dental structure.

- Occlusal, proximal and occluso-proximal restorations of medium and small sizes.

- Class III, IV and V restorations of anterior teeth .

- Composite resin direct veneers.

- Re-attachment of tooth fragments.

- Reducing and/or closing diastemas.

- Correcting and/or altering the form of one or several teeth.

- Structural defects: Amelogenesis Imperfecta, enamel hypoplasia, non-carious cervical lesions.

- Correcting and/or altering the proportional height and/or width of teeth.

- Deciduous teeth restorations.

- To correct and/or change the color of a part or the whole tooth.

#### Opalis shade system

Opalis uses the VITA® Classical shade guide for its shades of enamel and dentin. Besides, it also presents composites with shades of elevated translucence and opaque, and shades for bleached teeth and resin for reproducing value.

- The four translucent shades (T-Blue, T-Yellow, T-Orange e T-Neutral) allow the creation of esthetic restorations for application in very thin layers over the shades of the VITA® Classical guide, giving the dentist the possibility to reproduce the tooth structure perfectly.

A brief description of the application of translucent composites for enamel follows:

T-Blue: a resin of elevated translucence returning the opalescent effect to the tooth.

T-Yellow: a resin of elevated translucence reproducing the facial enamel and giving the tooth a depth aspect.

T-Orange: a highly translucent pigmented resin, bringing the natural opalescent and translucent effect of the enamel.

T-Neutral: a translucent resin which reproduces the palatal enamel and the incisal third. Its translucence is slightly inferior to the shades described above.

- The shades A0.5 and B0.5 can be used in odontopediatrics in order to reproduce with details the anatomy of deciduous teeth.

- The highly opaque shades, such as Opaque White and Opaque Pearl recreate details of the incisal halo and mask the sclerotic dentin.

- The shades of the VITA® Classical shade guide contemplate the dimension of chroma and shades of the resin. In order to complement the dimension of value, a thin layer of resin of value must be applied over the last layer of the restoration.

The resins of value (VH, VM, VL) bring luminosity to the restorations. The luminosity indicates the amount of white in the tooth. To reproduce such effect, we use the resins of value on the last layer. These are resins of elevated translucence which do not change the shades of the restoration.

When using the resin of value, consider that the thickness of the applied resin can bring an effect of high or low value to the restoration. The shades VM, VL and VL brings high, medium and low values to the restoration respectively.

- The composites for enamel of bleached teeth (E-Bleach H, E-Bleach M, E-Bleach L) and for dentin (D-Bleach) were developed for the necessity of restoring bleached and primary teeth. We offer one shade to reproduce the dentin and three shades for the enamel. The shade with lower saturation (brighter) is E-Bleach H

Table 1: Shade availability, translucency, quantity and exposure time.

Type	Shades	Quantity	Light curing	Translucence (%)	
Dentin	DA1	4 g	40s	43-46	
	DA2	4 g	40s	43-46	
	DA3	4 g	40s	43-46	
	DA3.5	4 g	40s	43-46	
	DB1	2 g	40s	43-46	
	DB2	4 g	40s	43-46	
	DB3	4 g	40s	43-46	
	DB4	4 g	40s	43-46	
	DC3	2 g	40s	43-46	
	D-Bleach	2 g	40s	43-46	
	E-Bleach H	2 g	40s	43-46	
	E-Bleach M	2 g	40s	43-46	
	E-Bleach L	2 g	20s	34-36	
	Enamel	EA1	4 g	20s	78.5
		EA2	4 g	20s	78.5
EA3		4 g	20s	78.5	
EA3.5		4 g	20s	78.5	
EB1		4 g	20s	53-56	
EB2		4 g	20s	53-56	
EB3		4 g	20s	53-56	
EC2		2 g	40s	66	
EC3		2 g	40s	66	
E-Bleach H		2 g	20s	78.5	
E-Bleach M	2 g	20s	78.5		
E-Bleach L	2 g	20s	66		
Effect Translucence	T-Blue	2 g	20s	78.5	
	T-Yellow			78.5	
	T-Orange			78.5	
T-Neutral		66			
Effect Opaque	Opaque Pearl(OP)	2 g	30s	35	
	Opaque White(OV)			35	
Value	VH (high)	2 g	20s	70	
	VM (medium)			70	
	VL (low)			70	

and the shade with higher saturation (darker) is E-Bleach L.

- The Opalis resin must be light cured layer by layer with a blue light and with a minimum intensity of 400 and 500nm with at least 450 mW/cm<sup>2</sup>. Do not remove the inhibiting layer as this affects the adhesion of the following layer of resin. The table 1 presents all shades of Opalis resin available, their respective translucence, commercialized amounts and light curing time.

- The light curing time described on table 1 must be followed to guarantee the curing depth, conversion degree, shade stability and mechanical properties of the resin. The capacity of the light to go through the resin is associated to translucence. As the Opalis resin presents different degrees of translucence, several studies were accomplished to determine the appropriate light curing time in order to achieve the resin properties and guarantee its quality. In case of insufficient exposure, the resin properties may not achieve the ideal values. Overexposure is not a possibility.

- Opalis presents the most chromatic shades and special shades in 2 to 3 syringes in order to cater for the needs of dentists.

#### Precautions and contraindications

- In case of allergic reaction discontinue the use.

- Avoid using lining or temporary materials based on eugenol as they hinder the light curing of the product.

- Avoid the contact of non-light cured Opalis with the skin, mucosa and eyes.

- Before light curing, the product may cause a slightly irritating effect and promote sensitivity due to the metacrylates.

#### Side effects

This product contains metacrylate monomers which may cause slight allergic or irritative reactions on patients who are sensitive to such substances.

#### Instructions for use

Read carefully the Warnings, Precautions, Contra-Indications and Possible Side Effects written in this Product information sheet, before the beginning of the treatment.

#### Shade taking

- Before taking the shade of the tooth to be restored, it must be clean and hydrated, without isolation. Prophylaxis is highly recommended in this step.

- The VITA® Classical or Opalis (FGM) shade guides may be used for shade taking.

- Alternatively, a small portion of the composite may be placed over the surface of the tooth to be restored (without etching nor application of an adhesive system), light cured for 30 seconds, humidified with the patient's saliva and observed. This stage may be accomplished to confirm the shade of the composite.

#### Product application

1. Isolate the operative field. The isolation must guarantee a humidity-free area which is clean and visible during the whole treatment. If necessary, perform the absolute isolation.

2. Prepare for the execution of direct adhesive restorations.

3. Acid etch (Condac 37% - FGM) the margins of the enamel initially and then inside the cavity. The acid etching time is 15 seconds. After that, wash with water abundantly, remove all the acid from the tooth structure and dry the dentin with oil dehydrating L.

4. Apply the adhesive system according to the manufacturer's instructions. If using a self-etching adhesive skip step 3.

5. Insertion and light curing of the composite resin.

- We recommend the technique of insertion and light curing by increments. Such technique allows a more uniform light curing. Follow the time described in Table 1 for layers of up to 1.5 mm of resin.

- Restore the cavity according to the selected shade(s) applying small layers of Opalis (maximum thickness of 1,5mm) adapting them in the cavity carefully.

6. For finishing and polishing, we recommend Diamond Pro sand disks and polish pastes Diamond I) and Diamond ACII, Diamond R or Diamond Exoc with the aid of felt disks Diamond and Diamond Flex (FGM).

**Note:** Checking occlusion is an extremely important procedure. Restorations in supra-occlusion cause excessive stress in the tooth and the restoration which in turn might cause pain. Do not leave them in intra-occlusion either.

#### Storage and maintenance

Keep the product in a fresh place. The syringes should be always tightly closed and protected. Keep out of direct sunlight. Store the product at temperature between 5°C and 30°C / 41°F and 86°F. Do not freeze the product.

#### Expiration date

Printed on the product packaging.

#### Warnings

All dental products should be used only by dentists, dental prosthetists and authorized laboratories. Do not use the product after it is expired.

When disposing the product, follow the legislation of your country. Keep out of the reach of children.

This material was manufactured for dental use only and must be manipulated according to the manufacturer's instructions of use. The manufacturer is not responsible for any damage caused by misuse or improper manipulation of the product. The user is obligated to prove (before using the product and under his/her responsibility) that this product is compatible with the desired utilization, especially when such utilization is not indicated in the instructions of use. Data description does not constitute any type of warranty and therefore is not binding.

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