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Desensitizer

Professional Use Only

Read all the information of these directions carefully before using the product. Keep them for consultation, at least until the product has been totally used or there is no more interaction of the product with your last patient.

Product Description

Desensibilize KF 0.2% is a desensitizing gel of low viscosity which basically has the neural action of the Potassium Nitrate and so does not interfere with the speed of the dental bleaching. It can be used before, during and after the bleaching. Moreover, with this concentration the patient can take it home.

It is indicated for treating low dental sensitivity before, during and/or after dental bleaching.

Forms of Presentation

Packaging containing one syringe with 2.5g of product and 1 disposable application tip.

Basic Composition

Active Ingredients: Potassium Nitrate at 5% and Sodium Fluoride at 0.2%.

Inactive Ingredients: Deionized Water, Glycerine, neutralizing agent and thickener.

Product Indication

It is indicated in the treatment of teeth with moderate to low sensitivity to temperature variances and food (dentinal Hyperaesthesia) and in the treatment of sensitivity caused by dental bleaching.

Precautions and Contraindications

- *Avoid excessive ingestion of the product on purpose.
- *Do not apply the product on gingiva in a poor conditions of health.
- *Do not use it on patients with anaemia and/or any sensitivity to the product components.
- *Discontinue using it if the patient has any undesirable reaction to the product components.
- *Not injectable. Do not reuse the syringe.

Side Effects

Ingestion of the product in large quantities can cause nausea and accentuate states of anaemia (effects of the Potassium Nitrate).

Instructions for Use

- 1)**Clean the surface of the teeth eliminating plaque, calculus and other deposits which can prevent the product from penetrating the teeth.
- 2)**Apply the desensitizing gel uniformly on the teeth (use a brush or Cavibrush micro-applier) and wait for at least 10 minutes. Alternatively the gel can be applied with the help of the bleaching tray, executing the application as if it were the bleaching agent itself.
- 3)**After the application time has ended, remove the gel from the teeth with cotton wool and plenty of water. Do not allow the patient to swallow the desensitizing agent.

4) Repeat the application of the desensitizing agent as required or, in the case of dental bleaching, following up the treatment time. The number of applications must not exceed once a day.

Note: After the application time of the desensitizing agent has ended and while it is still on the teeth, it is advisable to rub the product with the micro-applier, brush or felt disks at a low rotation in intermittent circular movements (low pressure). This procedure mechanically helps the desensitizing agent to penetrate the tooth and so enhances its performance.

If required, the patient can execute the application in his or her bleaching tray at home.

However, the patient must be fully guided about following the relevant safety instructions and not ingesting the product. In these cases it is important that the patient is fully aware of his or her responsibility.

Preservation and Storage

Keep the product in its original packaging always well sealed. Store the product at 41-86 °F. Do not freeze the product.

Warning

Do not use the product after expiry date. In order to discard the product, follow the legislation of your country. Keep out of reach of children.

This material was only manufactured for dental use and must be handled according to the instructions for use. The manufacturer is not responsible for damage caused by other uses or incorrect handling. Besides this, the user is obliged to prove, before use and under his or her responsibility, that this material is compatible with the use desired, mainly when this use is not indicated in the instructions for use. Data descriptions do not constitute any type of guarantee and so are not in way linked.

 DENTSCARE LTDA - Av. Edgar Nelson Meister, 474 Distrito Industrial - 89219-501 – Joinville – SC - Authorization of Functioning MS P5X44XY0XX28 CNPJ/Tax ID: 05.106.945/0001-06 - BRAZILIAN INDUSTRY - Registration at ANVISA Nº 80172310018 - Technical Authority: Friedrich Georg Mittelstädt - CRQ: 13100147-SC - Brand: FGM®

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