IDENTSCARE LTDA - Av. Edgar Nelson Meister, 474 - Distrito Industrial - 89219-501 - Joinville - SC - Authorization of Functioning MS P5X44XY0XX28 - CNPJ/Tax ID: 05.106.945/0001-06 - BRAZILIAN INDUSTRY - Registration at ANVISA Nº 80172319007 - Technical Authority: Friedrich Georg Mittelstädt CRQ: 13100147-SC - Brand: FGM®

Rev.: 00

Riscos à Saúde
Health Hazard
Peligro para la Salud

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Restaurador Dental

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Dental restorative material.

Professional Use Only

Read carefully all the information in this manual before using the product.

Keep it for later reference, at least until total consumption of the product and/or until there is no more interaction between the product and its last patient.

Product Description

ION-Z is a self-curing radiopaque glass ionomer based material. In order to produce long lasting restorations, the material presents high flexural and compressive strength as well as high bond strength and capability to avoid marginal leakage.

Presentations Form

Package containing:

- 01 Bottle of cement powder with 10g
- 01 Bottle of polycarboxylic acid with 8g
- 01 Powder scoop
- 01 Mixing pad
- Instructions for use

Basic Composition

Powder:

- Active ingredients: micronized glass ionomer
- Inactive ingredients: pigment (titanium dioxide and iron oxide), load (calcium-aluminum-zinc-fluoride silicate glass).

Liquid:

- Active ingredients: polycarboxylic and tartaric acids.
- Inactive ingredients: deionized water.

Product Indication

- Atraumatic Restorative Treatment (ART);
- Restorations of deciduous teeth (all classes);
- Provisional restorations of permanent teeth (all classes);
- Restorations of non-cariou cervical lesions;
- Base under permanent restorations (sandwich technique).

Precautions and Contraindications

- Do not apply the product directly on pulp. In deep cavities, it is indicated to use lining materials such as calcium hydroxide;
- Avoid contact of the product with the eyes. In case of undue contact, wash abundantly with water and seek medical attention;
- Avoid long contact of the liquid phase to the skin in order to avoid irritation. In case of undue contact, wash abundantly with running water.

Side Effects

Applying the product directly on pulp may lead to irritation and irreversible damage.

Use Instructions

1. Prepare the cavity removing only the affected dental tissue by means of rotary or manual instruments. Due to the high adhesive capacity of the product, there is no need for additional retentions.
2. With the aid of a sterile cotton pellet or disposable micro applicators (e.g. Cavibrush - FGM), apply one drop of the liquid on the cavity walls during 10 seconds. Then, wash the cavity and remove exceeding water

leaving the surface slightly moist. When dealing with deep cavities (risk of pulp exposure), apply calcium hydroxide lining before ION-Z;

3. The product presents two separate phases; liquid and powder. The correct proportion for mixing is 1:1, so one scoop of powder (scoop included in the kit) should be mixed to one drop of liquid for one portion to be obtained. The mixing should take place on a glass plate or mixing pad (included in the kit). Make sure not to have bubbles when dispensing the liquid because it could lead to adverse rheological properties;

4. With the aid of a plastic spatula, mix powder to liquid in small amounts gradually increasing until total mixing. The mixing time should be about 30 seconds;

5. Once mixed, the material presents itself shiny and with the ideal viscosity for proper application. With the aid of a nº1 spatula (or similar), apply ION-Z in the cavity in single or multiple increments while it is still shining. In case of deep cavities associated to single increment, make sure to avoid the presence of bubbles in the restoration. If needed, press the material against the cavity walls with your finger (gloved and with a small amount of Vaseline). The chemical curing time is about 3 to 4 minutes and during this step it is essential that the tooth remains isolated and free of contamination;

6. After chemical cure, remove excess of material and check occlusion to avoid premature contacts.

Notes:

- It is recommended to mix the product in temperatures between 15°C and 30°C (64°F to 86°F). Temperatures above this range shall shorten the working time, while temperatures below it might increase the time substantially.
- Mixing and setting times described in this manual can vary according to the local temperature, the mixing mode and the application mode.

Conservation and storage

Keep the bottles in their original packaging always properly closed. Store

the product on a dry place, on temperatures from 59°F to 86°F. Do not store the liquid in the refrigerator. It is not recommended to store the product together with other products containing eugenol, phenol or parachlorophenol.

Warnings:

Do not use the product after its expiration date. For disposal of the product, follow the legislation of your country. Keep out of the reach of children.